



Rapigen

One Step SARS-CoV-2 Antigen Rapid Test BIOCREDIT COVID-19 Ag Home Test Nasal

One step, rapid, immuno-chromatographic assay for the qualitative detection of SARS-CoV-2 antigen from Nasal swab

BIOCREDIT COVID-19 Ag Home Test Nasal is a rapid chromatographic immunoassay intended for the qualitative detection of nucleocapsid (N) protein antigen from SARS-CoV-2 in individuals with or without symptoms or other epidemiological reasons to suspect COVID-19 infection.

This test is authorized for home use with **self-collected** direct nasal (nares) samples using the sterilized nasal swab provided in the kit.

1st and officially approved by MFDS

- 1 Quick test result within 15-30 mins
- 2 Easy to use on site
- 3 Accurate test
- 4 User friendly Dual color system
- 5 Mutant SARS-CoV-2 detection



MFDS **IVD**

No compromise on quality

One Step SARS-CoV-2 Antigen Rapid Test

BIOCREDIT COVID-19 Ag Home Test Nasal

One step, rapid, immuno-chromatographic assay for the qualitative detection of SARS-CoV-2 antigen from Nasal swab

Rapigen

Assay Procedure

1 Preparation before test

1 Blow your nose (x3) 2 Wash your hands 3 Wear gloves

2 Assay diluent solution

1 Take out the assay diluent tube 2 Remove the seal 3 Insert the tube to the tube fixing box

3 Swab

1 Take out the swab 2 Collect specimen from both nostrils (x10) 3 Dilute and break (x10)

4 Filter cap

1 Insert the filter cap 2 Insert the tube to the tube fixing box

5 Test device

1 Take out the device 2 Specimen mixing (Gently! x10) 3 Dispense (15-30 min, 3 drops)

Interpretation of Results

Negative
One red "C" line within the result window
One red "C" line within the result window

Positive
Two bands ; black "T" line and red "C" line within the result window
Two bands ; black "T" line and red "C" line within the result window


Invalid
No "C" line within the result window. It is recommended that the specimen be retested using a new test device.
No "C" line within the result window.

Legend:
Negative: One red line at C, no line at T.
Positive: One red line at C, one black line at T.
Invalid: No red line at C, or one red line at C and one black line at T.

* If the black band appears (include smear) on the test line, contact the nearest screening clinic

Description of materials provided

Sterilized nasal swab, Filter cap, Assay buffer, Test device, Instructions for use, Plastic bag for disposal.

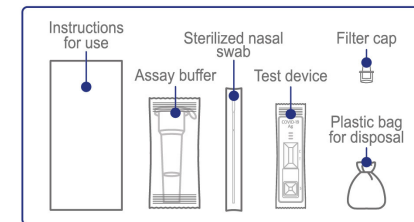

Manufactured by — **Rapigen, Inc.**

1152SNH073A00

161, Saneop-ro 155beon-gil, Gwonsong-gu, Suwon-si, Gyeonggi-do 16648, Republic of Korea
TEL : +82-31-427-4677 FAX : +82-31-427-4678 E-mail : info@rapigen-inc.com www.rapigen-inc.com